

All text and images can be edited

Intelligent Hearing Triage brought to you by Audiology-Online

Please touch the start button to begin.



START


Registration fields are editable too

Now touch the blue boxes below to register.

Your title:	Your first name:	Your last name:
Mr	Test	Client
Your Date of Birth: GP		
01	01	1930
Mr Mark Ashmore ▾		
		Next »

Cancel Test

The test starts



The test will take approximately five minutes.

Just answer some questions then listen to some sounds and touch a button each time you hear a sound.

Next »

Cancel Test

Referable conditions questions include an opportunity for the patient to expand on the answer

Have you had earache for longer than 7 days in the last 3 months?

Yes No

Cancel Test

Instructions to get the patient ready

Important.

If you wear hearing aids please remove them. Put the headphones on with the red ear cup on your right ear making sure the earcups completely cover your ears and touch the box to the right. Then touch Next.



Next »

Cancel Test

Headphone Orientation

When you press this button you should hear a sound in your **LEFT** ear.
If you are hearing the sound in your **RIGHT** ear reverse the headphones.

« Back

Next »

Cancel Test

[The triage process follows BSA recommendations for audiometry](#)

Test Instructions

We will test four sets of sounds in each ear. Before each set of sounds starts you will be asked to let us know if you hear a single sound by touching Yes or No. This enables us to set the correct level.

Then touch the **I Heard that** button each time you hear a sound. Don't worry if the sounds disappear for a short while. They will come back!

« Back

Next »

Cancel Test

Your test is now starting.

Can you hear a sound?

No

Yes

Cancel Test

Instructions

Now touch the button each time you hear a sound.

Testing...

I heard that

Cancel Test

The patient is warned when the tone is present to the other ear

We are now testing the other ear.
Now touch the button each time you hear a sound.


Testing...

I heard that

Cancel Test

A summary display is optional

Your test is completed



When you press the **Next** button your test results will be prepared, a summary displayed on the screen and a report sent to your GP.

Next »

Cancel Test

The output is produced immediately

Highlighted fields are mapped and therefore editable.

From: enquiries@GPTest.Test-Your-Hearing.co.uk [mailto:enquiries@GPTest.Test-Your-Hearing.co.uk]

Sent: 01 December 2013 20:18


To: mark@audiology-online.co.uk

Subject: Patient Hearing Test Results for Mr Test Client 01/01/1930

Dear Dr Ashmore,

One of your patients, Mr Test Client, dob 01/01/1930, has just completed the in-surgery hearing screening you requested and the results were as follows:

The test results for both ears are indicative of a moderate loss.

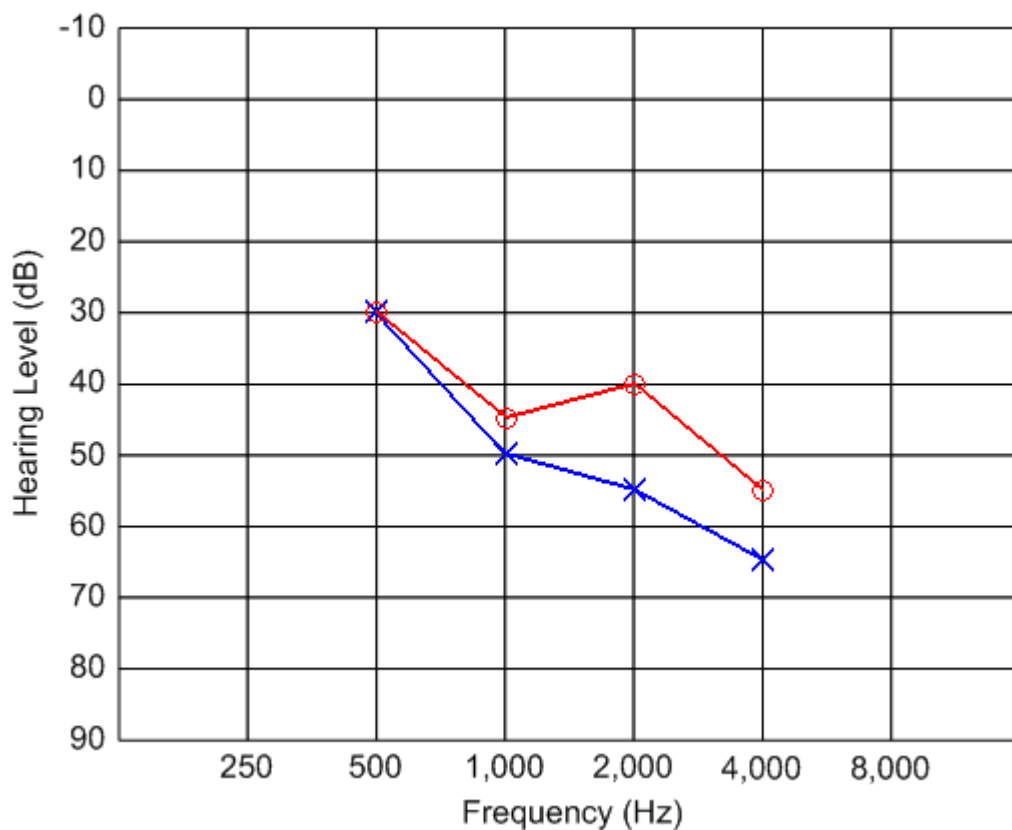

Your patient is aged 78. Subject to the audiogram recording a measurable loss and there being no contra indications below the patient should be referred to audiology.

Medical Conditions

1. The patient reported unilateral tinnitus. **They should be referred to ENT.**

Best regards

GPTest.Test-Your-Hearing.co.uk